

Covenant Health
Accu-Chek Inform II Competency Sheet

NAME:				UNIT: CSON	DATE:
				STUDENT ID:	
INITIAL TRAINING	6 MONTH	ANNUAL	☐		
Assessment	PASS	FAIL	METHOD		
HealthStream Interactive learning tool completed			certificate attached	1. Personal protection equipment must be worn when performing blood glucose or glucose control testing and when cleaning and disinfecting the Accu-Chek Inform II meter. True False	
Assemble testing supplies:					
Meter, strips, tote, QC solutions- high/low,			observed	2. The proper site for the finger puncture is on the side of the finger. True False	
alcohol wipes, 2X2s, gloves			observed		
Applies appropriate PPE - gloves			observed	3. The Accu-Chek Inform II meter should be docked in the base unit when not in use. True False	
Patient test/Simulated patient test					
Identifies patient using 2 patient identifiers			observed	4. Glucose control solutions do not have to be dated when first opened. True False	
Explains procedure to patient			observed		
Turns meter on, Scans operator ID			observed	5. Testing will start when the appropriate amount of sample is applied to the Accu-Chek Inform II test strip, about 0.6 ul. True False	
Verbalizes/carries out process if ID fails			observed		
Selects appropriate function (patient test)			obs/tchbk	6. Control solutions are stable for _____ months after opening? a. 2 months 4 months no expiration date 3 months	
Scans patient armband			observed		
Verifies test strip code matches vial			observed	7. What type of blood cannot be used with the Accu-Chek Inform II meter and the Accu-Chek Inform II test strips? a. Capillary Venous/Arterial Neonatal Cord blood	
Inserts strip, recaps vial immediately			observed		
Correct site care and specimen aquired			observed	8. You _____ scan the patient's armband instead of the patient's chart label to avoid errors. a. should should not	
Applies sample to front of strip correctly			observed		
Verbalizes/demonstrates response to abnormals/criticals (comments/ repeats)			obs/tchbk		
Confirms result/comment(s)			observed		
Cleans, performs, documents maintenance			observed		
*must stay "wet" for 1 minute			observed		
*not in strip port or on metal contacts			observed		
Can verbalize approved cleaning agents			tchbk		
Quality Control					
Verbalizes when QC must be performed			obs/tchbk		
Verbalizes LOCKOUT			obs/tchbk		
Verbalizes/Demonstrates high/low QC			obs/tchbk		
Verbalizes/demonstrates special docking station/use (and location if available)			obs/tchbk		
Results Review					
All results			observed		
Patient-specific results			observed		
Quiz Score:					
Pass / Fail					
This CSON student has satisfactorily completed the competency for the Accu-Chek Inform II bedside blood glucose monitor.					
Student Signature: _____					
Validator Signature: _____					
Validator's Printed Name: <u>Laiken Brazil, MSN, RN</u>					
This CSON student has NOT satisfactorily demonstrated competency. This form is being forwarded to _____ for further education/training. Date: _____ Student Signature: _____ Validator Signature: _____ Validator's Printed Name: _____					