



Set loading guide for the Alaris® Pump module


Alaris® System

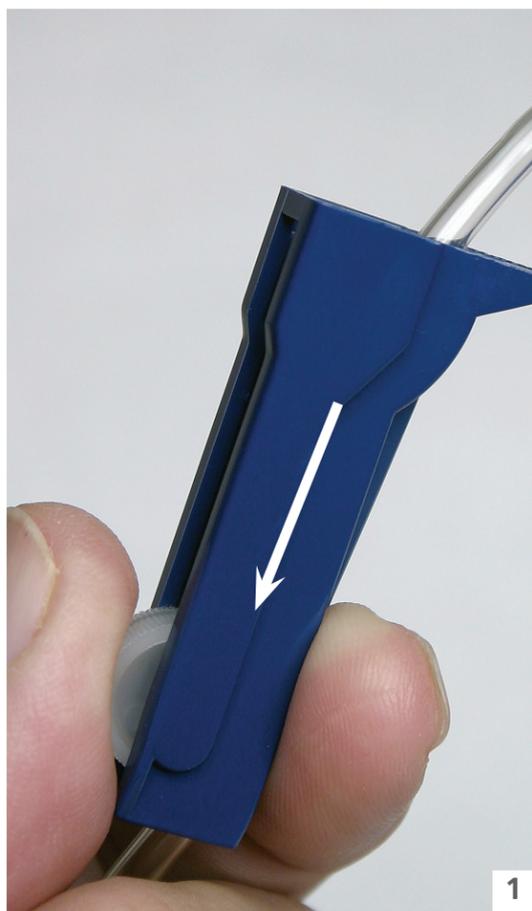

Prime set, close roller clamp

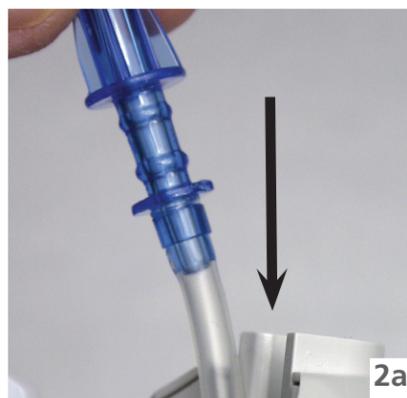

2a

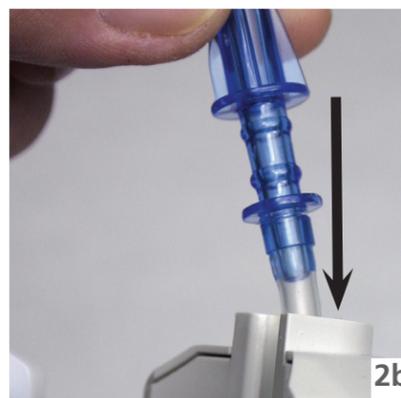


2b




2c

Hold the upper fitment above fitment recess and lower into the recess


Warning: Do not push or snap upper fitment snugly into upper recess

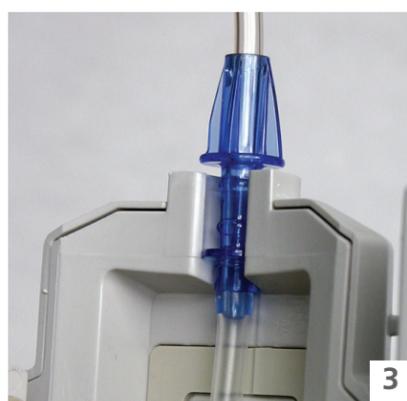

3

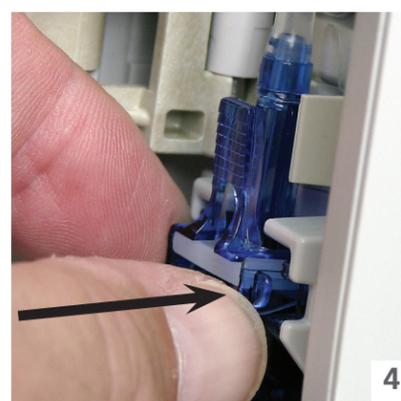

4

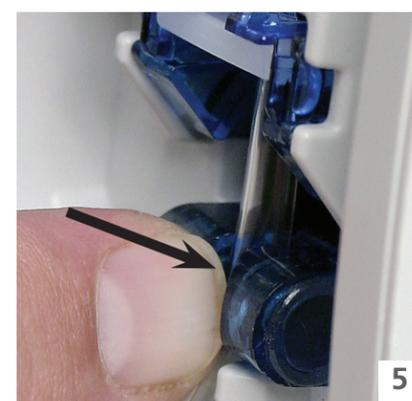

5

Grooves in fitment should fit square in upper fitment tubing retainer

Ensure that the tubing is not twisted

Firmly push tubing toward back of air-in-line detector



6



7



8

Do not touch set tubing

Close door and lower latch

Open roller clamp and verify that no fluid is flowing through the drip chamber

Examples of improper loading or foreign object in door.



Note: Unregulated flow or over-infusion may result if door is not fully closed.



9

Properly loaded upper fitment

For further information contact:
Customer Advocacy 800.854.7128

© 2010 CareFusion Corporation or one of its affiliates. All rights reserved. Alaris is a registered trademark of CareFusion Corporation or one of its affiliates. IF1019 (0110/1500)

CareFusion
San Diego, CA

carefusion.com

 CareFusion