

# Syringe delivery with the Alaris® Pump module



## Alaris® System



- Be sure the vent cap is in the closed position to prime the administration set. (Figure A)
- Remove the clear protective cover from the syringe adapter.
- Attach the prepared luer-lock syringe to the adapter. (Figure 1)



- Invert syringe and prime set by slowly pushing up on the syringe plunger until all air is removed from set. (Figure 2)

**NOTE: Approximate priming volume 3.8cc.**



- Close the blue slide clamp on the administration set.
- Remove blue protective cover from the pumping segment. (Figure 3)
- Install the set by positioning the upper fitment into the upper fitment recess, and then inserting the safety clamp fitment into clamp recess. Press the tubing into the air-in-line detector.
- Close the door—push in with one thumb on upper portion of module door while closing the door handle.



- Attach set to patient's vascular access device. **The plunger will not move during the infusion. The fluid will be displaced by air entering through the vent.**
- Program your Alaris® System as directed.
- **Open the white vent cap prior to initiating the infusion.** (Figure B)
- Open the blue slide clamp.
- press **Start**.



### NOTE:

- Close the white vent cap prior to re-priming the set. This action will prevent the vent material from wetting.
- Verify the white vent cap is open prior to resuming the infusion.

©2002-2007 Cardinal Health, Inc. or one of its subsidiaries. All rights reserved.  
Alaris® is a registered trademark of Cardinal Health, Inc. or one of its subsidiaries.  
3POC0492 (1007/2.5M)

♻️ Printed on recycled paper with 10% post-consumer fiber.

Cardinal Health  
San Diego, California

[cardinalhealth.com/alaris](http://cardinalhealth.com/alaris)