



# Attaching luer lock connectors to the SmartSite® needle-free valve

## SmartSite® Safety Disposables


### SmartSite® Needle-free Valve

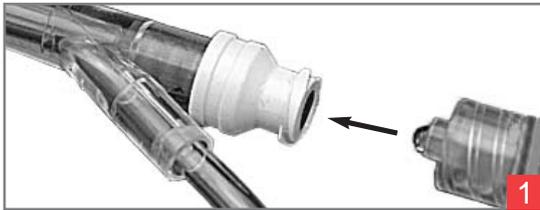

- Prior to every access, swab the SmartSite® needle-free valve port with 70% isopropyl alcohol (1-2 seconds) and allow to dry (approximately 30 seconds)
- Insert luer tip into the SmartSite® needle-free valve

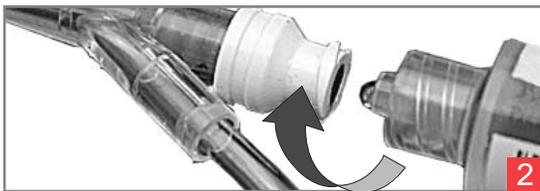

- Turn luer lock clockwise to engage

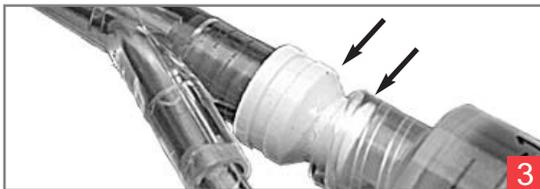

- Note **gap** indicating that the luer lock is not fully engaged



- Continue tightening luer lock until fully engaged to completely open the SmartSite® needle-free valve

Alaris® and SmartSite® are registered trademarks of Cardinal Health, Inc. or one of its subsidiaries.

© 2003-2005 Cardinal Health, Inc. or one of its subsidiaries. All rights reserved.  
3AD0274 (1205/2M)

♻️ Printed on recycled paper with 10% post-consumer fiber.

Cardinal Health  
Alaris® Products  
10221 Wateridge Circle  
San Diego, CA 92121

[www.cardinalhealth.com/alaris](http://www.cardinalhealth.com/alaris)